INSTRUCTIONS FOR USE

Quadrant Universal LC

Quadrant Universal LC is a light-cured, fluoride releasing radiopaque Microglass composite for universal application. The particle size of the glass filler renders it polishable to a very high gloss...

- Composition
Quadrant Universal LC is based on a Bis-GMA matrix and contains 60% filler by volume or 72% by weight, which is:
• Ba-Al-F-silicate glass (0,02-2 µm)
• highly dispersed silicon dioxide (0,02-0,07 µm)

- Indications
• restorations of class I,II,III,IV and V cavities, e.g.:
• reshaping
• shade correction
• direct veneers
• splinting of traumatized teeth

Interactions with other materials
Do not use in combination with eugenol containing restoratives, as these impair the polymerisation.

Use of Quadrant Syringes
Turn the cap from the syringe. Turn the plunger clockwise to express material from the syringe. After use, turn the plunger back for 1/4 revolution to release the pressure from the material.

Use of Quadrant Capsules
Place the capsule into the barrel of the Q-Gun. Remove cap from capsule. Squeeze by squeezing the handle with a slow, steady pressure.

- Clinical procedure
1. Prepare the cavity according to the general directions for composite restorations. Clean and dry the cavity. The use of rubberdam is recommended. When restoring exposed root surfaces, a preparation may not be necessary.
2. Apply Quadrant Uni-SE-Bond, Quadrant UniBond or Quadrant Uni-1-Bond (or an other universal bonding system as desired) following the instructions for use for that product.

Table with 2 columns: Shade, Polymerization time per 2 mm (s). Rows include A1, A2, A3, A3.5, B2, B3, C2, C3, I, A4, OA2.

Table with 2 columns: Kleur, Polymerisatietijd per 2 mm (s). Rows include A1, A2, A3, A3.5, B2, B3, C2, C3, I, A4, OA2.

Safety Advice
Quadrant Universal LC contains monomers capable of polymerization (e.g. multi-functional (meth)acrylic acid esters), which may cause irritations in particularly sensitive individuals.

Storage advice
Do not store at temperatures higher than 25°C (77°F). Do not use this material beyond its expiry date.

Made by Cavex Holland BV
For dental use only.

Our technical advice, whether verbal, in writing or by way of trials, is given in good faith but without warranty, and this also applies where proprietary rights of third parties are involved.

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